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#### **A. Intended use**

Cito NovaClean 50% is a strong disinfection solution (High level) based on citric acid for disinfecting and decalcifying machine hydraulic system.

- Wide effective range of virus, bacteria, fungi and spores removal.
- Unique decalcification properties without corrosion.
- Unpleasant odorless than similar products

#### **B. Composition**

Citric acid, deionized water

#### **C. Warning and Precautions**

1. Inspect the container before use.
2. The solution inside the canister should be transparent and its packaging should be healthy, otherwise it should not be used.
3. Store Cito NovaClean 50% at 1-30 °C.
4. Avoid from drinking and inhaling of solution.
5. In case of direct contact with the Cito NovaClean 50% rinse the contacted place immediately with plenty of water.
6. In case of accidental ingestion of Cito NovaClean 50%, see a doctor.
7. The solution can be used for 30 days, if canister's cap left open.
8. Wearing gloves and mask is recommended during the use of Cito NovaClean 50%.

#### **D. Instruction for use**

To use Cito NovaClean 50%, follow the instruction of manufacturer of the hemodialysis machines for heat disinfection with citric acid based disinfectant, or follow the recommends of hemodialysis machine maintenance engineers.

#### **E. Manufacturer**

Name: Nova pharmed chemia

Factory: 2nd Bustan, West Hafez Blvd, Eshtehard, Industrial Zone, IRAN

Office: Fifth floor, No. 44, Saadat Abad Ave., 1998836841, Tehran, IRAN

Tel: +98(21)220 74001 Fax: +98(21)220 73103

Email: [info@Novapharmed.co](mailto:info@Novapharmed.co)

Website: [www.Novapharmed.co](http://www.Novapharmed.co)

#### **F. Symbols**

Symbol					
Indication	Consult Instruction for use	Lot number	Expiry date	Caution	Storage condition

<i>Doc. Code</i>	<i>Rev. No.:</i>	<i>Date Of Issue</i>	<i>Page</i>
F72300RD08	R00	98/02/21(11/May/2019)	2 of 1